
digene® HC2 DNA Collection Device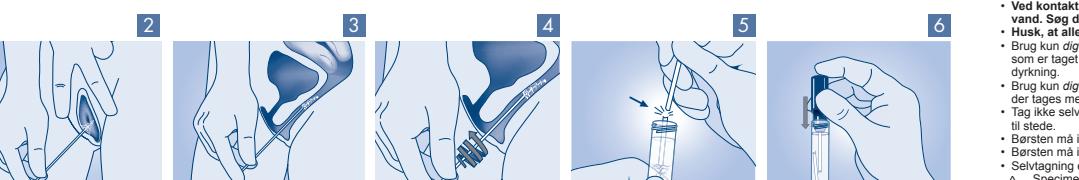

EN Name and Intended Use

The digene HC2 DNA Collection Device is intended for the collection and transport of physician-collected cervical specimens to be tested only with the digene Hybrid Capture® 2 (HC2) HPV and CT/GC DNA Tests and self-collected vaginal specimens to be tested only with the digene HC2 High-Risk HPV DNA Test®. THIS DEVICE IS NOT INTENDED FOR PAP COLLECTION.

For best results, users must first read these instructions before beginning collection and then perform each step in the listed order. For Storage Instructions, Specimen Transport, or the physician-collection procedure, see the physician-collected instructions for use in the distribution box.

Reagents and Materials Provided

1x1 BRUSH Brush

1x1ml STM Specimen Transport Medium (contain 0.05% sodium azide)

Warnings and Precautions

- Specimen Transport Medium:** Warning! Causes mild skin irritation. Wear protective gloves/ protective clothing/ eye protection/ face protection. Immediately wash affected area of skin with water and soap, and rinse thoroughly. See reverse side for Specimen Collection and Handling instructions.
- Do not reuse.** Do not use if package is damaged.
- Do not use cervical brush with pregnant women. Please let your physician know if you think you may be pregnant.
- Do not put in your mouth or ingest Specimen Transport Medium (tube liquid). If you do put the tube liquid in your mouth or ingest, contact a physician immediately.
- Do not inhale (breathe in) Specimen Transport Medium (tube liquid). If you do inhale the tube liquid, get fresh air; consult a physician in case of complaints.
- If contact with eyes occurs, rinse opened eye for several minutes under running water. Then consult a doctor.
- Consider all human specimens to be potentially infectious.
- Use only the digene HC2 HPV DNA and digene HC2 CT/GC DNA tests for physician-collected cervical specimens. Specimens collected with this system cannot be used for culture.
- Use only the digene HC2 High-Risk HPV DNA Test for self-collected vaginal specimens. Specimens collected with this system cannot be used for culture.
- Do not collect self-collected vaginal specimens if antifungal cream or contraceptive jelly are present.
- Do not touch the brush before or after the specimen is collected.
- Do not touch the brush to any surface outside of the collection tube.
- Specimen Transport Medium contains sodium azide, which may react with lead or copper plumbing to form highly explosive metal azide compounds. Dispose of in accordance with local regulations. Contact your local QIAGEN Representative or your local medical professional on how to dispose or clean spills.

Storage Instructions

- Store kits at 15–30°C. Do not use after the expiration date indicated next to the symbol ☰ on the pouch label.
- Specimen Collection and Handling**
- Vaginal Self-Collection Procedure**

- Unscrew or pull off the cap from the tube and place it down on a hard flat surface with the inside of the cap pointing up. Be careful not to spill the Specimen Transport Medium (liquid in the tube).
- Gently open the outside of your vagina and insert the brush into the opening (Step 2).
- Continue to insert the brush by gently rotating the brush to the left or to the right until you meet resistance (Step 3). If you experience pain or major discomfort, stop and consult your nurse or physician.
- Gently hold the brush in place. Count to 10 slowly before rotating. Rotate the brush 5 times in a circle to collect the sample (Step 4).
- Remove the brush from your vagina and put it in the provided tube (brush facing down) until it touches the bottom. Break the end of the handle by snapping it at the line on the handle while holding the tube firmly with one hand and placing the other hand near the end of the brush (Step 5). Upon breaking the handle, discard the broken portion.
- Only the lower portion, with the brush side down, should be in the tube and immersed in the Specimen Transport Medium (liquid in the tube). Secure cap (Step 6), clearly label the tube, and then give the tube to a medical professional.

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